

# FMRIB 7T Participant Emergency Evacuation

In an emergency situation, the tabletop and participant must be removed from the magnet bore.


## Removing Participant with Table Controls



- When power supply and/or motorised drive are intact, the Home Button is the fastest method for moving the tabletop out of the magnet bore.
- Press the Home Position Button to move the tabletop completely out of the magnet bore. The Speed button can be used to increase or decrease the table speed.

## Removing Participant Manually

- In case the Table Stop button has been depressed, power failure and/or defective motorised drive,
- Pull the emergency release handle fully outwards
- The tabletop is mechanically decoupled from the motorised drive unit.
- Pull the tabletop out of the magnet using the handle at the foot end.

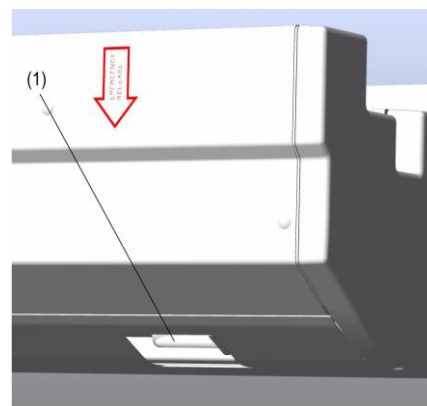

## MRI Table to Trolley Transfer



- In the event of a medical emergency, slide the participant down the bed and out of the head coil
- Place the trolley next to the table with the sides down
- **Apply the trolley breaks**
- Position sliding board to bridge the gap between table and trolley
- Slide participant onto the trolley
- Release brakes and raise sides before evacuating

